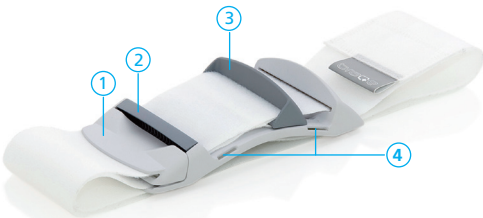
PATIENTS FEEL MORE SECURE AND SUPPORTED DURING THEIR RECOVERY PERIOD.

Contents
User guide, Shoulder straps, BracePlus™.
1 Buckle 2 Brake 3 Handle 4 Holes (for shoulder straps)

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EN

BracePlus™ is a post-operative sternal support band designed to assist patient recovery following cardiothoracic surgery. The easy-to-adjust sternal support band ensures that the chest and healing sternum are supported at all times without restricting normal breathing. BracePlus™ can be used with confidence throughout your hospital recovery and at home following hospital discharge.

Usage
BracePlus™ can be worn throughout your recovery and should be worn over your normal clothes. BracePlus™ can be worn throughout the day, except when showering or bathing.

Care
BracePlus™ should not be washed. For hygiene reasons BracePlus™ should not be used by multiple users. BracePlus™ is made from Polyamide.

Indications
BracePlus™ is a post-operative sternal support band. Used for rehabilitation after open heart surgery.

Contra Indication
Use BracePlus™ only use for what it is intended for and not as a medical binder.

Refer to operating instructions	Single Use Only	Do not wash	Do not iron
Manufactured by	Batch No.	Catalogue No.	

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